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en Usage instructions - Heat pack

Introduction

The pack with a heat-dispensing filling aids relaxation and gives a feeling of wellbeing. It can be used externally for heat treatment.

Advice Read the safety instructions carefully, and only use the product as described in these instructions, in order to avoid injuries or damage. Store these usage instructions carefully, so that you can re-read them if necessary. If the product is passed on to someone else, these instructions must also be included.

WARNUNG or **CAUTION**

Scope of delivery

• Heat pack. Available with different fillings, such as

- Rape seeds
- Cherry stones
- Grape stones
- Silica gel beads
- Ceramic beads

• Usage instructions

The packaging can be reused or recycled.

Safety information

Please note that the manufacturer assumes no liability for damage or accidents that are caused by non-observance of the usage instructions.

Intended use

The pack is suitable for promoting relaxation through heat treatment. The pack is exclusively intended for domestic and not for commercial usage.

Foreseeable misuse

Do not use the pack in the case of febrile diseases, diminished neural sensitivity or temperature perception issues. Please note: Children are sensitive to heat. The pack is also not designed to be used in the case of extensive skin lesions or skin disorders. If you are unsure, consult your doctor or pharmacist.

Safety instructions

- The pack is only intended for external application to undamaged skin. The pack filling must not be eaten.
- The pack must be checked for wear-and-tear or damage before each use. Only undamaged products should be used. The pack will be damaged if it comes into contact with hot, pointed or sharp surfaces, or with oil, grease or sunlight.
- The heating process should only be carried out by an adult.
- Children may only use the pack under adult supervision.
- Persons with restricted physical or mental capacities and persons who cannot sense heat correctly should only use the pack under supervision.

Warming the pack in the microwave

- Product** damage can result from **overheating!**
- Do not use the pack if it is damaged.
- Always keep a half-filled glass of water in the microwave with une pack. If the pack has an outer cover, it should be removed before warming.
- Do **not** warm the pack by placing it in a grill or in a container of hot water.
- Never place it on a metal surface to warm it.
- Only use a microwaveable dish.
- The microwave rotary plate must be clean and able to rotate freely.
- Follow the microwave operating instructions.
- Follow the warming instructions.

- Lay the pack flat in the microwave. You may need to fold larger packs once in the middle. Spread the filling out evenly.
- Set the microwave in accordance with the settings on the care label or the 'Technical data' table.
- After warming, knead the pack well to spread the temperature out.
- Test the temperature of the pack on your forearm.

If the pack hasn't reached the desired temperature, repeat steps 1-4 once more.

If the pack is already too hot, let it cool down to room temperature and check it for any damage.

The warmed pack should not be left on any part of the body for more than 20 minutes.

Heating in the oven

max. 10 minutes at 80 °C.

Warning for burns

- If the pack is too hot, this can cause burns. Be aware that burns can occur at temperatures over 42 °C, depending on the duration of use.
- Allow your body enough time to recover before placing a warm pack on it again.
- Remove the pack immediately if the heat becomes uncomfortable.

Care, storage and disposal advice

- Store in a dry place.
- Protect the pack against exposure to direct sunlight.
- Do not wash the inner pack. If necessary, just wipe it with a damp cloth. If there is an outer cover, it can be washed in accordance with the care label.
- Do not leave any objects on the pack.
- For reasons of hygiene, the pack should be replaced three years after the date of purchase. The product can be disposed of as domestic rubbish.

Technical data			
	Heat pack		
Warming the pack in the microwave (no grill function)	<ul style="list-style-type: none">with rape seed filling with cherry stone filling with grape stone filling	with silica gel beads	with ceramic bead filling
600 – 800 watts	60 seconds (max. twice)	100 seconds (max. twice)	100 seconds (max. twice)
800 – 1000 watts	40 seconds (max. twice)	80 seconds (max. twice)	80 seconds (max. twice)

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